

ATLANTIS™ Abutment

Abutment screw torque recommendations

The torque recommendations are references for your convenience.
They might change without further notice in order to follow manufacture's recommendations.

Manufacturer	Implant	Torque Ncm
BioHorizons	BIOHORIZONS	
	Internal 3.5, 4.5, 5.7	30
	Tapered Internal 3.5, 4.5, 5.7	30
	External implant 3.5	30
	External implant 4.0, 5.0	30
Biomet 3i	BIOMET 3i	
	Certain™ 3.25, 4.0, 4/3, 5.0, 5/4, 6.0	20
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4, 5/6/5, 4/3, 5/4, 6/5	20
	Certain™ XP 4/5, 5/6	20
	MicroMiniplant™ 3.25	20
	Miniplant™ 3.25	20
	Osseolite™ 3.75, 4.0, 5.0, 6.0	20
	Osseolite™ XP 4/5, 5/6	20
Camlog	CAMLOG	
	Screw-line Implant 3.3	20
	Screw-line / Root-line Implant 3.8, 4.3, 5.0, 6.0	20
DENTSPLY Implants	DENTSPLY IMPLANTS	
	ANKYLOS® C/X 3.5, 4.5, 5.5, 7.0	15
	OsseoSpeed™ / OsseoSpeed™ TX 3.0 – Yellow	15
	OsseoSpeed™ / OsseoSpeed™ TX 3.5, 4.0 – Aqua	20
	OsseoSpeed™ / OsseoSpeed™ TX 4.5, 5.0 – Lilac	25
	OsseoSpeed™ 4.0 - Blue (Previous)	25
	OsseoSpeed™ TX Profile 4.5, 5.0	25
	XIVE® S 3.0	24
	XIVE® S 3.4, 3.8, 4.5, 5.5	24
	FRIALIT™ 3.4, 3.8, 4.5, 5.5	24
	Keystone Dental	KEYSTONE DENTAL
PrimaConnex™ SD 3.3, 3.5, Genesis™ 3.8		30
PrimaConnex™ RD 4.0, 4.1, Genesis™ 4.5		30
PrimaConnex™ WD 5.0, Genesis™ 5.5, 6.5		30
Restore™ SD 3.3, 3.4		30
Restore™ RD 3.75, 4.0		30
Restore™ WD 5.0, 5.5, 6.0		30
Nobel Biocare	NOBEL BIOCARE	
	NobelActive™ NP 3.5	35
	NobelActive™ RP 4.3, 5.0	35
	NobelReplace™ NP 3.5	35
	NobelReplace™ RP 4.3	35
	NobelReplace™ WP 5.0	35
	NobelReplace™ 6.0	35
	Brånemark System™ NP 3.3	35
	Brånemark System™ RP 3.75, 4.0	35
	Brånemark System™ WP 5.0, 5.5	35
	SteriOss Replace™ 3.1, 3.5, 4.3, 5.0, 6.0	35
	SteriOss HL™ 3.25, 3.8, 4.5, 5.0, 6.0	35

Please turn over ➡

ATLANTIS™ Abutment

Abutment screw torque recommendations

Manufacturer	Implant	Torque Ncm
Straumann	STRAUMANN	
	Bone Level 3.3 NC (Narrow CrossFit™)	35
	Bone Level 4.1, 4.8 RC (Regular CrossFit™)	35
	Standard/Standard Plus 4.8 RN (Regular Neck)	35
	Standard/Standard Plus 4.8 WN (Wide Neck)	35
	Standard Plus 3.3 NN (Narrow Neck)	35
Zimmer Dental	ZIMMER DENTAL	
	Tapered Screw-Vent™ 3.5, 4.5, 5.7	30
	Screw Vent™ 3.3, 3.7, 4.5	35
	SwissPlus™ 4.8	35

All trademarks and companies name are the property of their respective owners.

Medical and legal disclaimer

The information (including torque recommendations) is provided for information purposes only and does not constitute advice for medical or treatment purposes, nor a legal contract, between DENTSPLY Implants and any person or entity. Although all reasonable effort is made to present accurate information (including torque recommendations), DENTSPLY Implants makes no guarantees of any kind, including but not limited to that it may be outdated, invalid or subject to debate. The information is provided “as is” without warranty of any kind, either express or implied, including without limitation, the implied warranties of merchantability, fitness for use of a particular purpose, or non-infringement.

You agree that neither DENTSPLY Implants, nor any other company within the DENTSPLY international group, will be liable to you for any loss or damages, either actual or consequential, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants. In particular, DENTSPLY Implants will have no liability for any consequential, indirect, punitive, special or incidental damages, whether foreseeable or unforeseeable, arising out of or relating to these terms, or to your (or any third party's) reliance upon information (including torque recommendations) obtained from DENTSPLY Implants, whether based in contract, tort, statutory or other law, except only in the case of death or personal injury where and only to the extent that applicable law requires such liability.